

# PRODUCT INFORMATION

Date: 28/1 - 2008	Product info.: 2-2008	Written by: Fredrik Delås
Item: Seat cushion Uno Seat	Delivery: From this date	

## Seat cushion Uno Seat will replace Netti Single.

Seat cushion Uno Seat is produced by Alu Rehab and will replace Netti Single January/February 2008. When we have no more Netti Single in stock, we will start sending out Uno Seat.

### Build up:

Uno Seat is always ready for use! Depending on the size of the cushion Uno Seat is fitted for different user weights.


The cushion is built from four layers of Polyurethane foam – with different properties and weight carrying capacity. Two of the layers are placed next to each other horizontally in the middle layer. At the rear there is used a softer foam than in front. This is to maintain a stabilization in front of the seat bones and to prevent sliding forward in the seat. On top there is used a foam to allow quick sink in into the cushion, and at the same time creates a large as possible contact area for optimum pressure distribution. The bottom layer is made of a firmer foam to avoid sitting through the cushion.

### Outer cover:

Alu Rehab supplies Uno Seat with Mars or EasyCare outer covers. The Mars cover is very stretchable and allows a high degree of air transportation. This is the same material as used on Netti Single today. The EasyCare cover is similar to TechmaFlex used on the Single cushion. It has an easy to clean surface which protects the core from pollution.

### Documentation:

Uno Seat has gone through thorough testing including biocompatibility and fire retardancy. The properties of the cushion have been tested at Duncan Bain Consulting showing very good results.

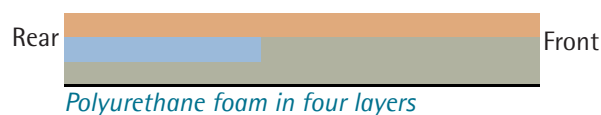

Simple to use



Stabilizing



Pressure distribution

**ALU REHAB**<sup>TM</sup>  
Thinking ahead - moving forward

Alu Rehab AS  
Bedriftsveien 23  
NO-4353 Klepp Stasjon  
Norway