Netti | Uno Seat

Netti | Sit

Deutsch	Bedienungsanleitung
English	Operation Instructions
Italiano	Istruzioni d'uso
Français	Mode d'emploi
Norsk	Brukerveiledningen
Dansk	Brugsanvisning
Svenska	Bruksanvisning

Deutsch

INHALT

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English

CONTENTS

1. Purpose of product
 2. Range of use
 3. Technical specifications
 4. Potential risk
 5. Mounting description
 6. Maintenance
 7. Routines for inspection
 8. Washing/Reuse
-

1. Purpose of product

Netti|Uno Seat and Netti|Sit are seat cushions developed to fit all types of wheelchairs. They are built on the concept of simplicity in use in combination with good pressure ulcer preventive properties. The pressure ulcer preventive properties cover both pressure distribution and micro climate.

A combination of a friction textile and velcro under the cushion makes it compatible with all existing wheelchairs on the market.

2. Range of use

Netti|Uno Seat is a simple pressure ulcer preventive cushion built from different layers of polyurethane foam for long time pressure distribution, with integrated support area in

front of the seat bones. It comes with a 3D outer cover offering increased ventilation and micro climate control.

The cushion is well suited for pressure ulcer prevention related to long time seating.

Netti|Sit is a pressure ulcer preventive cushion built from different layers of polyurethane foam for long time pressure distribution. It has side supports for increased stability and contact area. An integrated abduction block offers a good foundation for erection of the upper trunk. A cut surface in the top foam layer reduces surface tension and allows quick sink in. Integrated support area in front of the seat bones.

The cushion is well suited for pressure ulcer prevention related to long time seating, and where one is seeking stability for the user.

3. Technical specifications

Netti|Uno Seat

CORE

4 layers polyurethane foam. The cushion is made from polyurethane with different weight carrying properties. Two of the layers are mounted next to the other horizontally in the middle section. At the rear there is used a softer polyurethane foam than in the front. This is to assure stabilization in front of the seat bones.

OUTER COVER

100% knitted polyester with 3D-structure. The cover is very durable and stretchable with a high degree of air penetration.

The bottom textile of the outer cover is made from a friction fabric with mounted velcro, securing the cushion to stay in position in any wheelchair model and on any surface.

Netti|Sit

CORE

4 layers polyurethane foam. The cushion is made from polyurethane with different weight carrying properties. Two of the layers are mounted next to the other horizontally in the middle section. At the rear is used a softer polyurethane foam than in the front. This is to assure stabilization in front of the seat bones.

On top of the lower weight carrying foam layer, there is mounted a textile binding the surface of the foam. This gives possibility for reduction of total weight and increase of weight carrying capacity.

It has side supports for increased stability and contact area and an integrated abduction block for assistance for erection of the upper trunk.

INNER COVER

The core has a white protective cover, ProTex, protecting the core from pollution. Protex is a semi permeable "breathing" textile made from polyurethane. It lets vapour through, while water and other liquids are prevented from entering the core. ProTex is very elastic.

OUTER COVER

Model 1: 3D

100% knitted polyester with 3D-structure. The cover is very durable and stretchable with a high degree of air penetration.

Model 2: 3D + °C

100% knitted polyester with 3D-structure + °C – PCM (Phase Change material). This is a textile stabilized at 32 °C, which means it isolates and releases heat when temperatures are moving above and under this level. The purpose of the °C-cover, is to postpone/reduce the heating process of tissue and skin.

The bottom textile of the outer cover is made from a friction fabric with mounted velcro, securing the cushion to stay in position in any wheelchair model and on any surface.

4. Potential risk

The cushions are "pre-fit", and needs no more fitting as long as mounting- and washing instructions are followed. If the user has open ulcers, we recommend the user not to sit on the cushion.

5. Mounting description

The cushion is taken out of the packing and placed in the seat of the wheelchair. Place the cushion according to instructions on the label attached to the bottom fabric of the outer cover. Check that the velcro and friction fabric is facing down towards the seat plate. The front of the cushion should be even with the front of the seat plate.

6. Maintenance

The only need for maintenance of the cushions is cleaning:

1. Open the zipper at the rear and take the outer cover off.
2. Open the zipper at the bottom of the middle white ProTex cover (Netti|Sit).
3. Take out the core, and place it edgewise for airing while the covers are washed.
4. Follow washing instructions described on label attached to the bottom of the middle and outer cover.
5. The core is placed in the covers with the label facing down. For correct position of front and rear, see label.
6. Make sure that the label and zipper of the middle cover (if used) is facing down.
7. Mount the outer cover. Make sure that the zipper faces rearwards and that the label faces down.

7. Routines for inspection

Before seating the user, check that the cushion is placed correct according to instruction on label under the cushion. On a regular basis, check that the core and covers are placed correctly (follow description under chapters 5 & 6).

The need for washing core & covers will vary from user to user depending on degree of incontinence etc. A rule of thumb can be to take the core out of the covers once a week and let it stand edgewise to aerate and at the same time wash the covers.

It is recommended to have an extra cover for the cushion so that the user doesn't need to wait for the covers to dry.

The core and the covers are to be washed separately. Use mild detergent. **DO NOT USE CHLORINE BLEACH.** After washing, shake the core to remove excess water and let it stand to dry. Do not use harsh cleaners such as scouring pads & heavy-duty grease removers, or solvents, such as acetone. To remove spots and stains, use standard household cleansers and a soft cloth. To loosen heavy, dried-on soil, you may first need to saturate the spot.

8. Washing / Reuse

	Netti Uno Seat	Netti Sit
CORE		
Wash	Hand wash 40 °C	Hand wash 40 °C
Disinfection	Virkon S*	Virkon S*
	Auto clave 105 °C	Auto clave 105 °C
Drying	Squeeze	Squeeze
	Air dry standing edgewise	Air dry standing edgewise
INNER COVER	Not standard	
Wash		Machine wash 95 °C
Drying		Tumble dry max 85 °C
OUTER COVER		
Wash	Machine wash 60 °C	Machine wash 60 °C
Drying	Tumble dry max 85 °C	Tumble dry max 85 °C

* Read instructions on package.

Alu Rehab considers seat cushions to be an intimate and very personal technical aid. If reusing, we advice to at least exchange the outer and also the inner covers.

Alu Rehab AS
Bedriftsveien 23
NO-4353 Klepp stasjon
Tel: (+47) 51 78 62 20
post@alurehab.com

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