

SPARE PART CATALOG NETTI SVIPP

Version 1/11 - English

INDEX

| | Page |
|---|------|
| Seat frame and backrest | 3 |
| Seat cover - adjustable front | 4 |
| Seat frame and backrest | 5 |
| Backrest | 6 |
| Seat cushion - complete | 7 |
| Seat cushion - outer cover | 8 |
| Backrest - recline kit | 9 |
| Push handle | 10 |
| Head support rack | 11 |
| Head support A | 12 |
| Head support B | 13 |
| Head support C | 14 |
| Arm support left | 15 |
| Arm support right | 16 |
| Extension piece foot support | 17 |
| Universal foot supports - complete | 18 |
| Universal foot supports - parts | 19 |
| Angle adjustable foot supports - complete | 20 |
| Angle adjustable foot supports - parts | 21 |
| Bearing house | 23 |
| Front castors | 24 |
| Brake | 25 |
| Main wheels | 26 |
| Anti tip | 27 |


Produced by:

Alu Rehab AS, Bedriftsveien 23, NO-4352 Klepp stasjon, Norway

Phone: + 47 51 78 62 20 - Fax + 47 51 78 62 21


E-mail: post@alurehab.com

www.alurehab.com

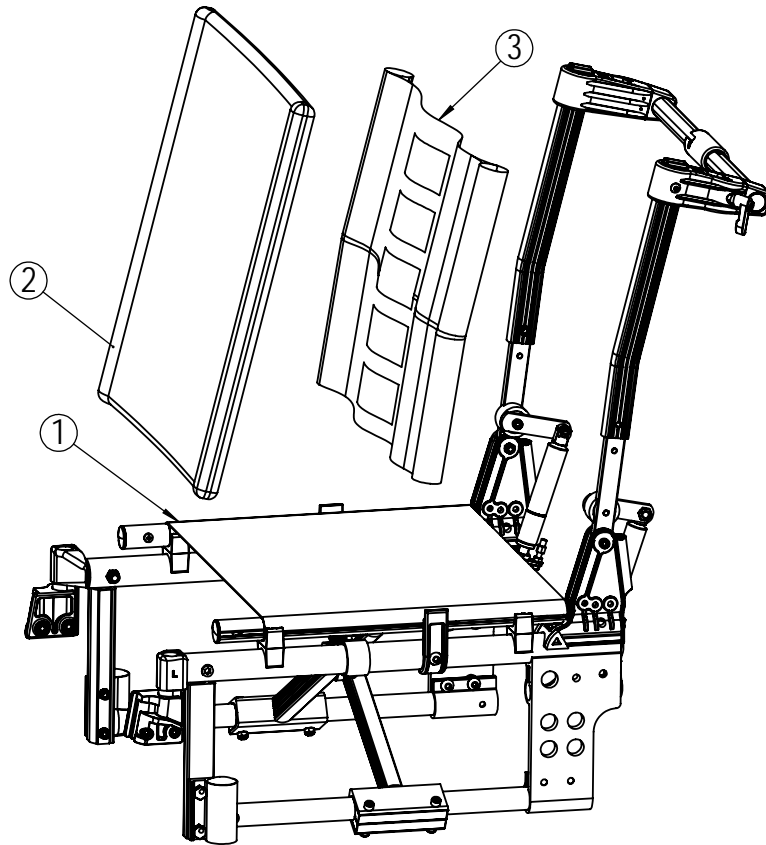


Seat frame and backrest

| Position | Description | Item no. | Pcs. pr. chair |
|----------|---|----------|----------------|
| 1 | Cross Seat cover 35cm Netti Svipp long | 22020 | 1 |
| 1 | Cross Seat cover 38cm Netti Svipp long | 22021 | 1 |
| 1 | Cross Seat cover 40cm Netti Svipp long | 22022 | 1 |
| 1 | Cross Seat cover 43cm Netti Svipp long | 22023 | 1 |
| 1 | Cross Seat cover 45cm Netti Svipp long | 22024 | 1 |
| 1 | Cross Seat cover 50cm Netti Svipp long | 22025 | 1 |
| 1 | Cross Seat cover 55cm Netti Svipp long | 22026 | 1 |
| 1 | Cross Seat cover 35cm Netti Svipp short | 22170 | 1 |
| 1 | Cross Seat cover 38cm Netti Svipp short | 22171 | 1 |
| 1 | Cross Seat cover 40cm Netti Svipp short | 22172 | 1 |
| 1 | Cross Seat cover 43cm Netti Svipp short | 22173 | 1 |
| 1 | Cross Seat cover 45cm Netti Svipp short | 22174 | 1 |

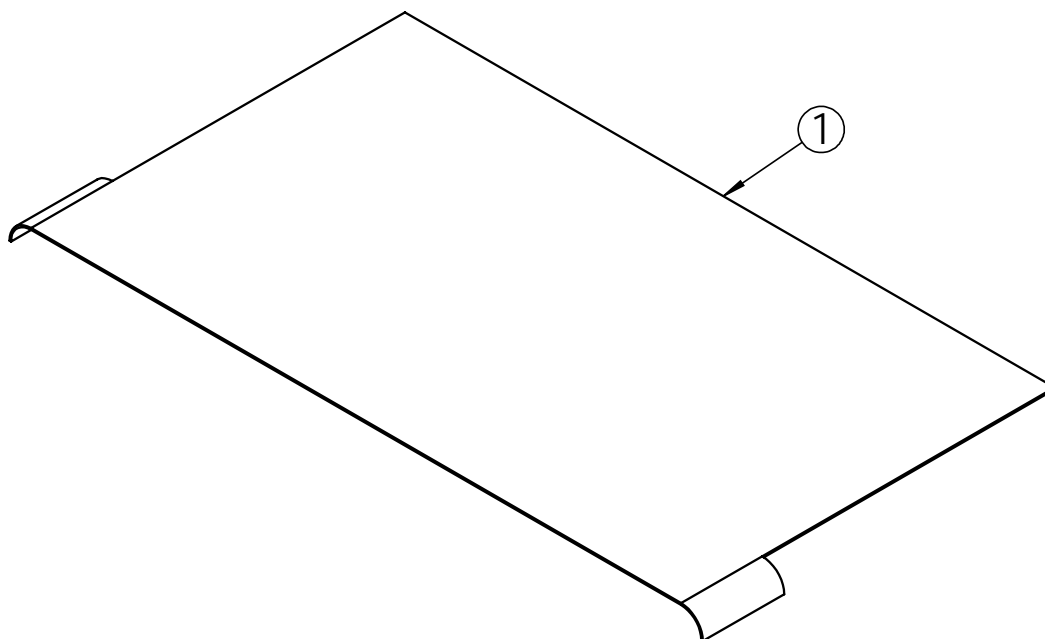


Seat cover - adjustable front

| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Seat cover adjustable front Netti Svipp 35cm | 22391 | 1 |
| 1 | Seat cover adjustable front Netti Svipp 38cm | 22392 | 1 |
| 1 | Seat cover adjustable front Netti Svipp 40cm | 22393 | 1 |
| 1 | Seat cover adjustable front Netti Svipp 43cm | 22394 | 1 |
| 1 | Seat cover adjustable front Netti Svipp 45cm | 22395 | 1 |
| 1 | Seat cover adjustable front Netti Svipp 50cm | 22396 | 1 |
| 1 | Seat cover adjustable front Netti Svipp 55cm | 22397 | 1 |

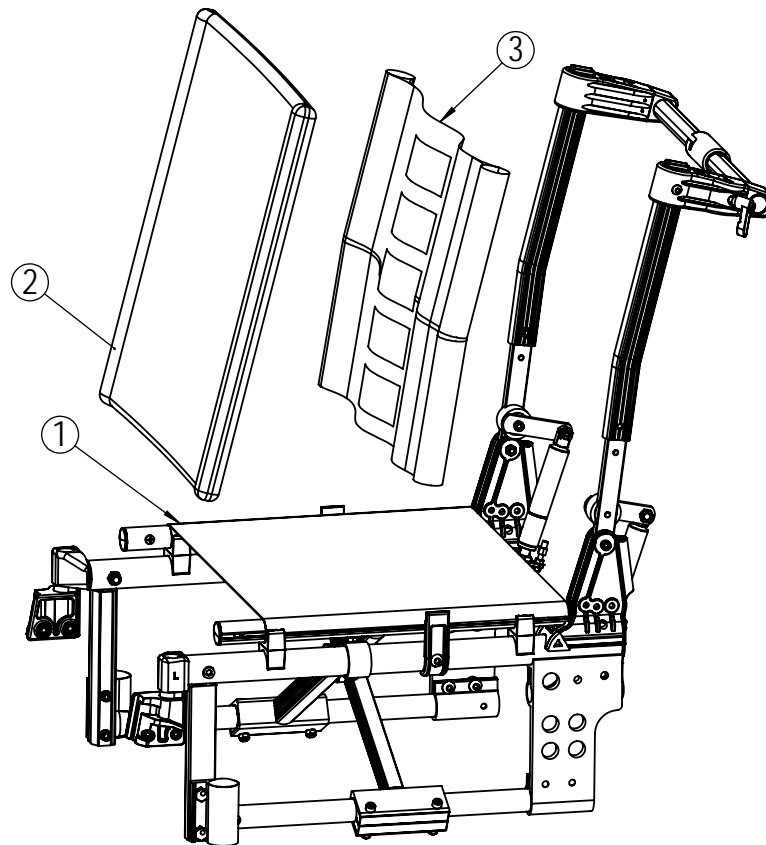

Seat frame and backrest


| Position | Description | Item no. | Pcs. pr. chair |
|----------|---------------------------------------|----------|----------------|
| 2 | Backrest cushion Flip Mars 35cm | 22052 | 1 |
| 2 | Backrest cushion Flip Mars 38cm | 22053 | 1 |
| 2 | Backrest cushion Flip Mars 40cm | 22054 | 1 |
| 2 | Backrest cushion Flip Mars 43cm | 22055 | 1 |
| 2 | Backrest cushion Flip Mars 45cm | 22056 | 1 |
| 2 | Backrest cushion Flip Mars 50cm | 22057 | 1 |
| 2 | Backrest cushion Flip Mars 55cm | 22058 | 1 |
| 3 | Backrest Velcro 35-50 cm Height: 35cm | 20699 | 1 |
| 3 | Backrest Velcro 55-60 cm Height: 35cm | 2865 | 1 |

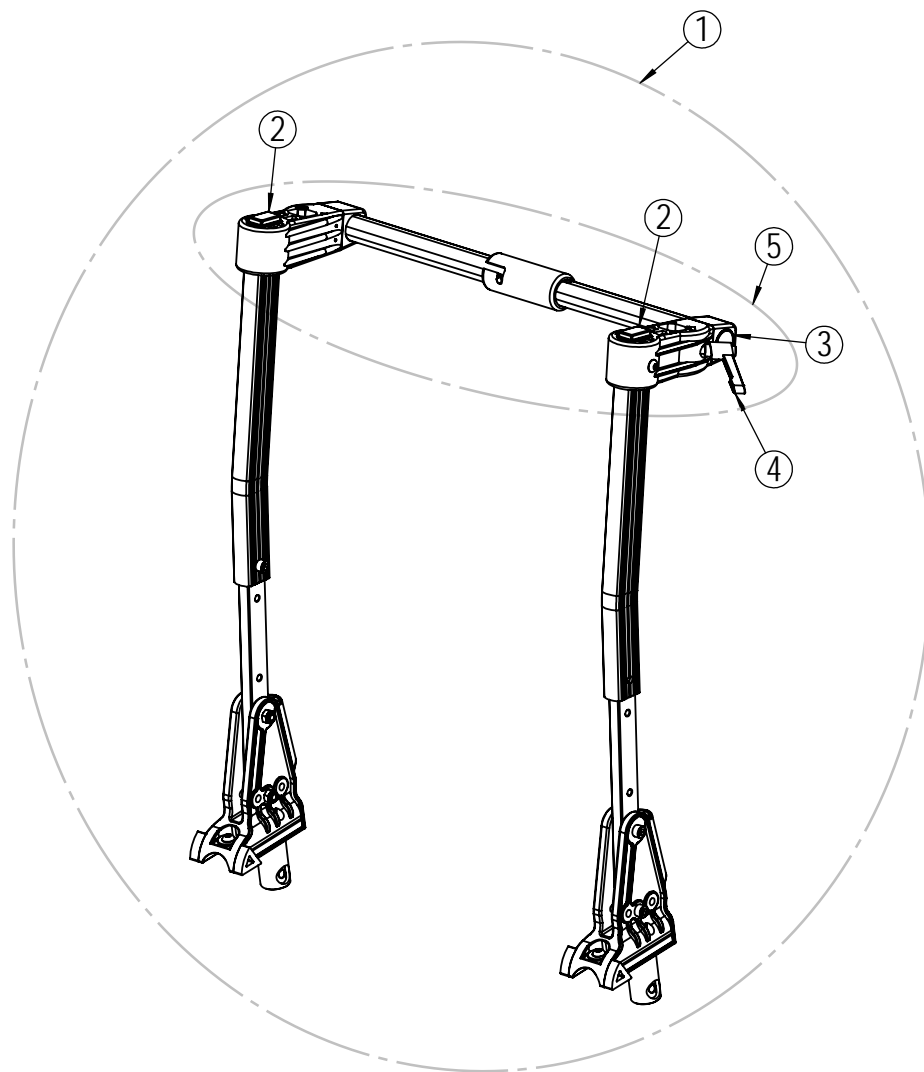

Backrest


| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Backrest Netti Svipp 35cm komplett | 26125 | 1 |
| 1 | Backrest Netti Svipp 38cm komplett | 26126 | 1 |
| 1 | Backrest Netti Svipp 40cm komplett | 26127 | 1 |
| 1 | Backrest Netti Svipp 43cm komplett | 26128 | 1 |
| 1 | Backrest Netti Svipp 45cm komplett | 26129 | 1 |
| 1 | Backrest Netti Svipp 50cm komplett | 26130 | 1 |
| 1 | Rygg Netti Svipp 55cm komplett | 26131 | 1 |
| 2* | Square plug 25x15 | 21847 | 2 |
| 3* | Plastic plug | 20509 | 2 |
| 4* | Clamp lever M6x16 | 20097 | 2 |
| 5 | Backrest stabilization bar complete 35cm | 23419 | 1 |
| 5 | Backrest stabilization bar complete 38cm | 23420 | 1 |
| 5 | Backrest stabilization bar complete 40cm | 23211 | 1 |
| 5 | Backrest stabilization bar complete 43cm | 23421 | 1 |
| 5 | Backrest stabilization bar complete 45cm | 23422 | 1 |
| 5 | Backrest stabilization bar complete 50cm | 23423 | 1 |
| 5 | Backrest stabilization bar complete 55cm | 23424 | 1 |

* Available upon request

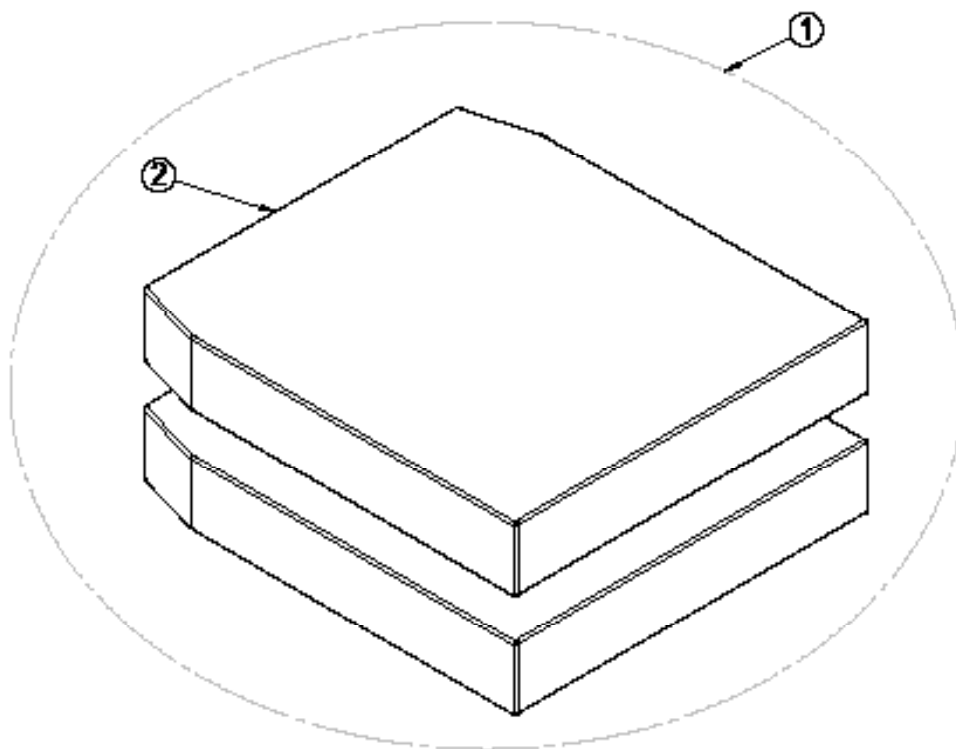


Seat cushion - complete

| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Seat cushion, Uno Seat Width:35cm Depth:40cm, Mars | 840377 | 1 |
| 1 | Seat cushion, Uno Seat Width:35cm Depth:45cm, Mars | 840378 | 1 |
| 1 | Seat cushion, Uno Seat Width:35cm Depth:50cm, Mars | 840379 | 1 |
| 1 | Seat cushion, Uno Seat Width:38cm Depth:40cm, Mars | 840380 | 1 |
| 1 | Seat cushion, Uno Seat Width:38cm Depth:45cm, Mars | 840381 | 1 |
| 1 | Seat cushion, Uno Seat Width:38cm Depth:50cm, Mars | 840382 | 1 |
| 1 | Seat cushion, Uno Seat Width:40cm Depth:40cm, Mars | 840385 | 1 |
| 1 | Seat cushion, Uno Seat Width:40cm Depth:45cm, Mars | 840386 | 1 |
| 1 | Seat cushion, Uno Seat Width:40cm Depth:50cm, Mars | 840387 | 1 |
| 1 | Seat cushion, Uno Seat Width:43cm Depth:40cm, Mars | 840390 | 1 |
| 1 | Seat cushion, Uno Seat Width:43cm Depth:45cm, Mars | 840391 | 1 |
| 1 | Seat cushion, Uno Seat Width:43cm Depth:50cm, Mars | 840392 | 1 |
| 1 | Seat cushion, Uno Seat Width:45cm Depth:40cm, Mars | 840395 | 1 |
| 1 | Seat cushion, Uno Seat Width:45cm Depth:45cm, Mars | 840396 | 1 |
| 1 | Seat cushion, Uno Seat Width:45cm Depth:50cm, Mars | 840397 | 1 |
| 1 | Seat cushion, Uno Seat Width:50cm Depth:40cm, Mars | 840400 | 1 |
| 1 | Seat cushion, Uno Seat Width:50cm Depth:45cm, Mars | 840401 | 1 |
| 1 | Seat cushion, Uno Seat Width:50cm Depth:50cm, Mars | 840402 | 1 |
| 1 | Seat cushion, Uno Seat Width:55cm Depth:40cm, Mars | 840405 | 1 |
| 1 | Seat cushion, Uno Seat Width:55cm Depth:45cm, Mars | 840406 | 1 |
| 1 | Seat cushion, Uno Seat Width:55cm Depth:50cm, Mars | 840407 | 1 |

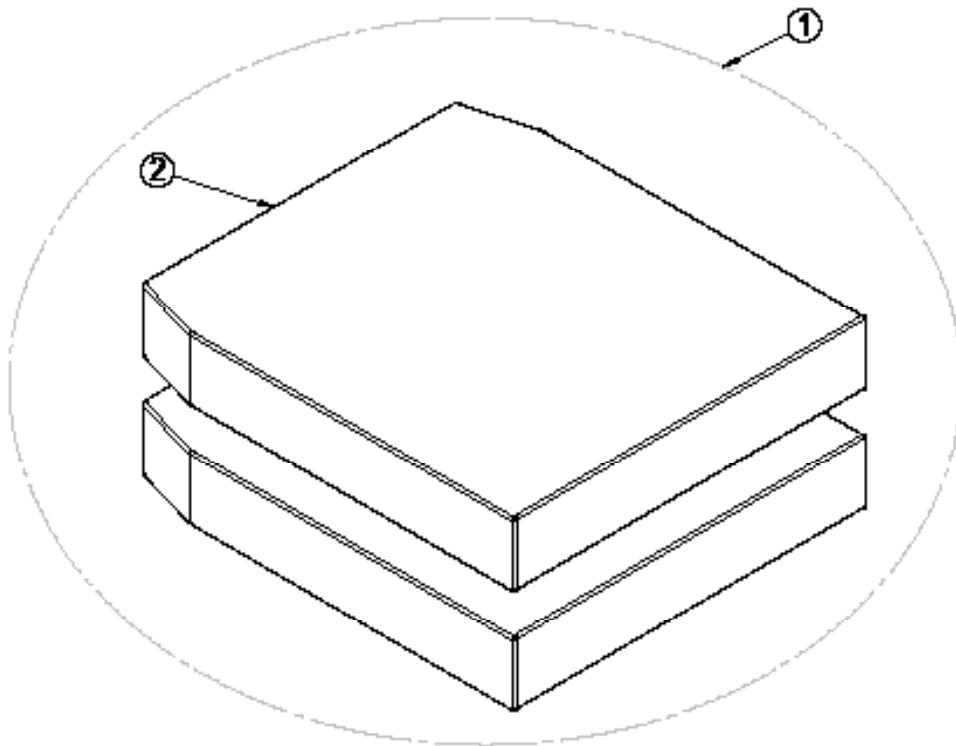

Seat cushion - outer cover


| Position | Description | Item no. | Pcs. pr. chair |
|----------|---|----------|----------------|
| 2 | Outer cover, Uno Seat Width:35cm Depth:40cm, Mars | 3543540 | 1 |
| 2 | Outer cover, Uno Seat Width:35cm Depth:45cm, Mars | 3543545 | 1 |
| 2 | Outer cover, Uno Seat Width:35cm Depth:50cm, Mars | 3543550 | 1 |
| 2 | Outer cover, Uno Seat Width:38cm Depth:40cm, Mars | 3543840 | 1 |
| 2 | Outer cover, Uno Seat Width:38cm Depth:45cm, Mars | 3543845 | 1 |
| 2 | Outer cover, Uno Seat Width:38cm Depth:50cm, Mars | 3543850 | 1 |
| 2 | Outer cover, Uno Seat Width:40cm Depth:40cm, Mars | 3544040 | 1 |
| 2 | Outer cover, Uno Seat Width:40cm Depth:45cm, Mars | 3544045 | 1 |
| 2 | Outer cover, Uno Seat Width:40cm Depth:50cm, Mars | 3544050 | 1 |
| 2 | Outer cover, Uno Seat Width:43cm Depth:40cm, Mars | 3544340 | 1 |
| 2 | Outer cover, Uno Seat Width:43cm Depth:45cm, Mars | 3544345 | 1 |
| 2 | Outer cover, Uno Seat Width:43cm Depth:50cm, Mars | 3544550 | 1 |
| 2 | Outer cover, Uno Seat Width:45cm Depth:40cm, Mars | 3544540 | 1 |
| 2 | Outer cover, Uno Seat Width:45cm Depth:45cm, Mars | 3544545 | 1 |
| 2 | Outer cover, Uno Seat Width:45cm Depth:50cm, Mars | 3544550 | 1 |
| 2 | Outer cover, Uno Seat Width:50cm Depth:40cm, Mars | 3545040 | 1 |
| 2 | Outer cover, Uno Seat Width:50cm Depth:45cm, Mars | 3545045 | 1 |
| 2 | Outer cover, Uno Seat Width:50cm Depth:50cm, Mars | 3545050 | 1 |
| 2 | Outer cover, Uno Seat Width:55cm Depth:40cm, Mars | 3545540 | 1 |
| 2 | Outer cover, Uno Seat Width:55cm Depth:45cm, Mars | 3545545 | 1 |
| 2 | Outer cover, Uno Seat Width:55cm Depth:50cm, Mars | 3545550 | 1 |

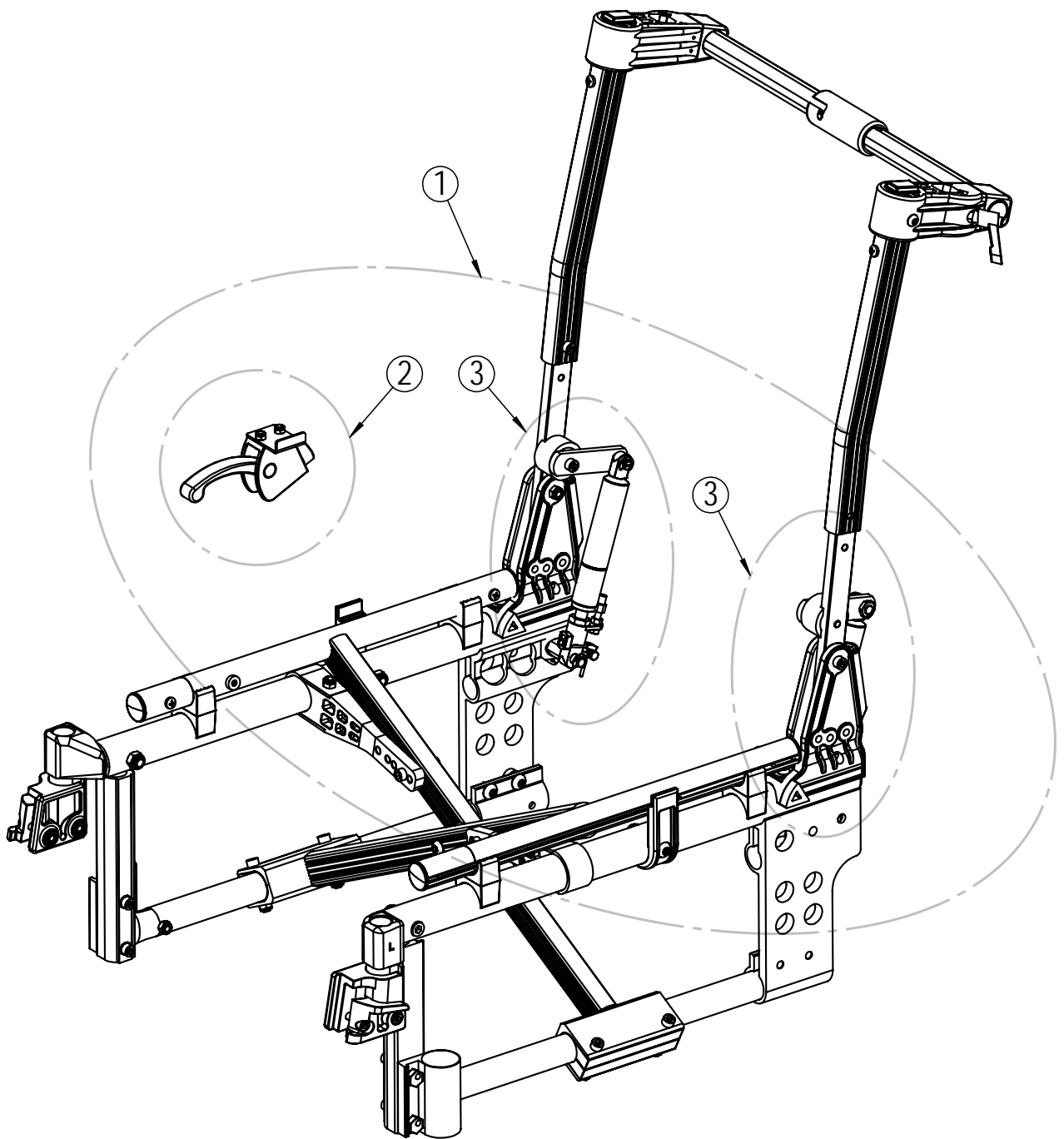

Backrest- recline kit*

| Position | Description | Item no. | Pcs. pr. chair |
|----------|---|----------|----------------|
| 1 | Recline kit complete | 25400 | 1 |
| 2 | Release handle recline right | 26190 | 1 |
| 2 | Release handle recline left | 26191 | 1 |
| 3 | Gas cylinder recline Netti Svipp complete | 24471 | 2 |


* Available upon request

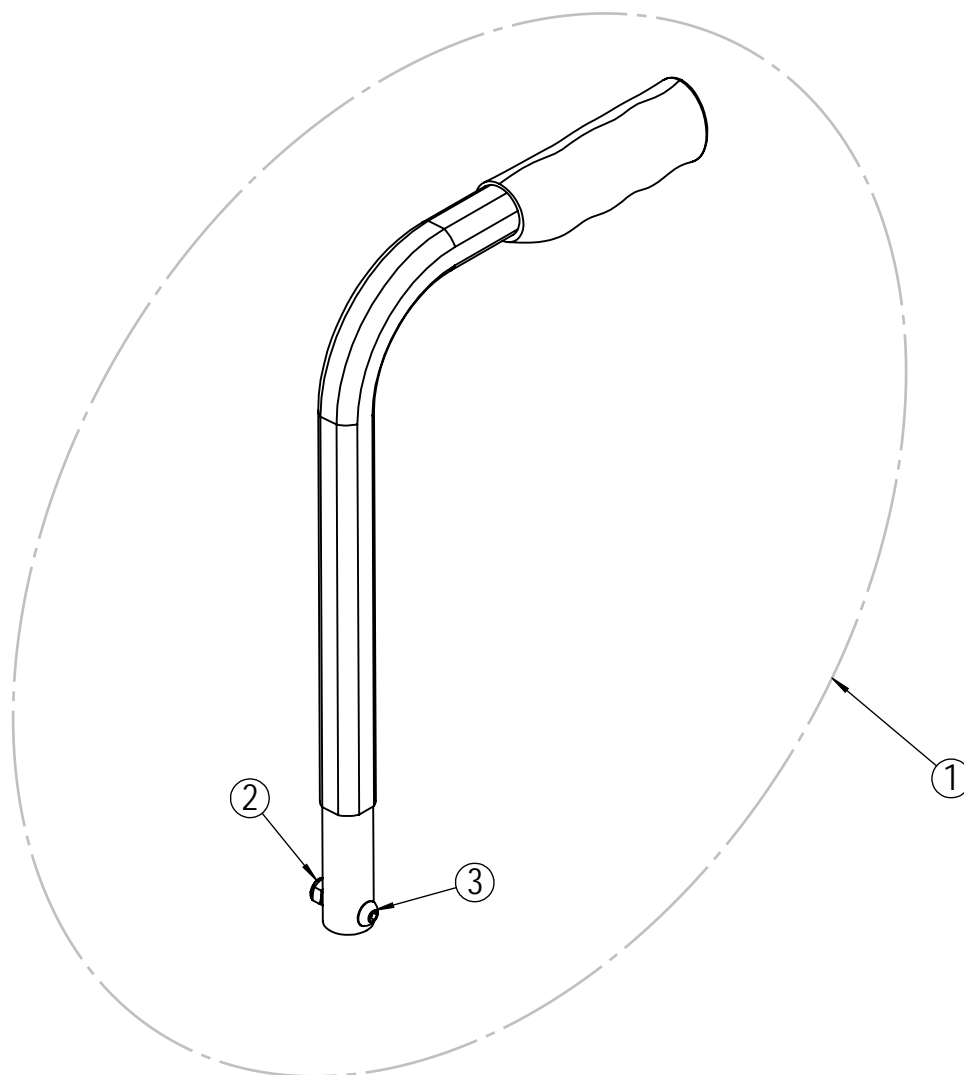

Push handle


| Position | Description | Item no. | Pcs. pr. chair |
|----------|-------------------------------------|----------|----------------|
| 1 | Push handle standard 300mm complete | 21754 | 2 |
| 1 | Push handle long 500mm complete | 21808 | 2 |
| 2* | Lock nut M6 | 20058 | 2 |
| 3* | Screw M6x25 | 20992 | 2 |

* Available upon request

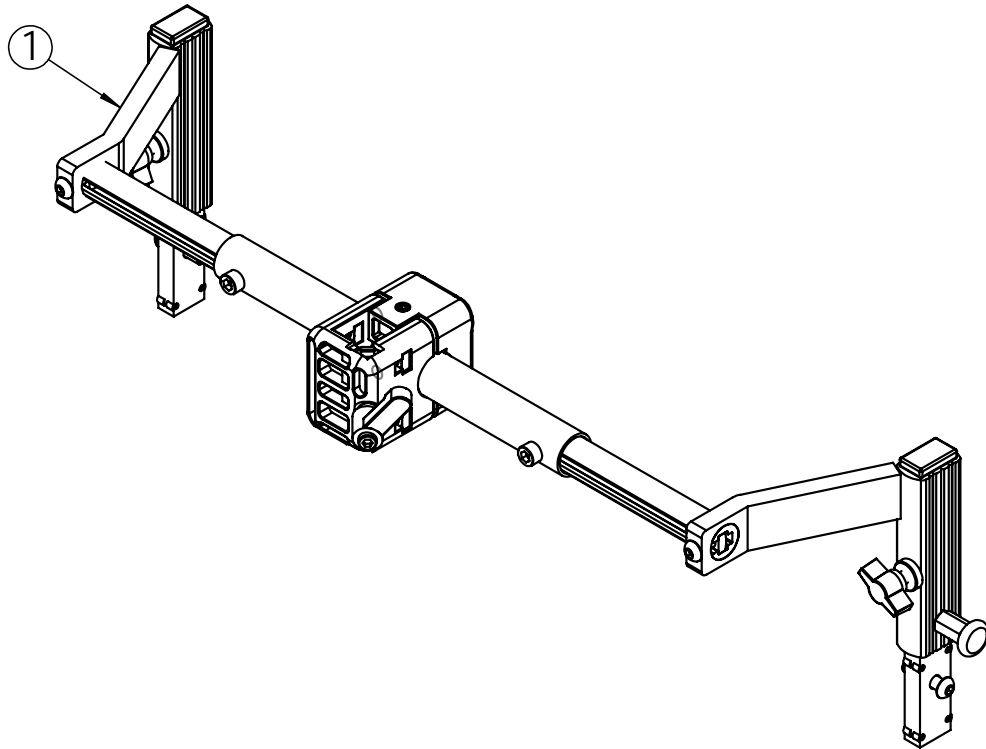

Head support rack


| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Head support rack Netti Svipp 35-45cm complete | 27214 | 1 |
| 1 | Head support rack Netti Svipp 50-55cm complete | 27215 | 1 |

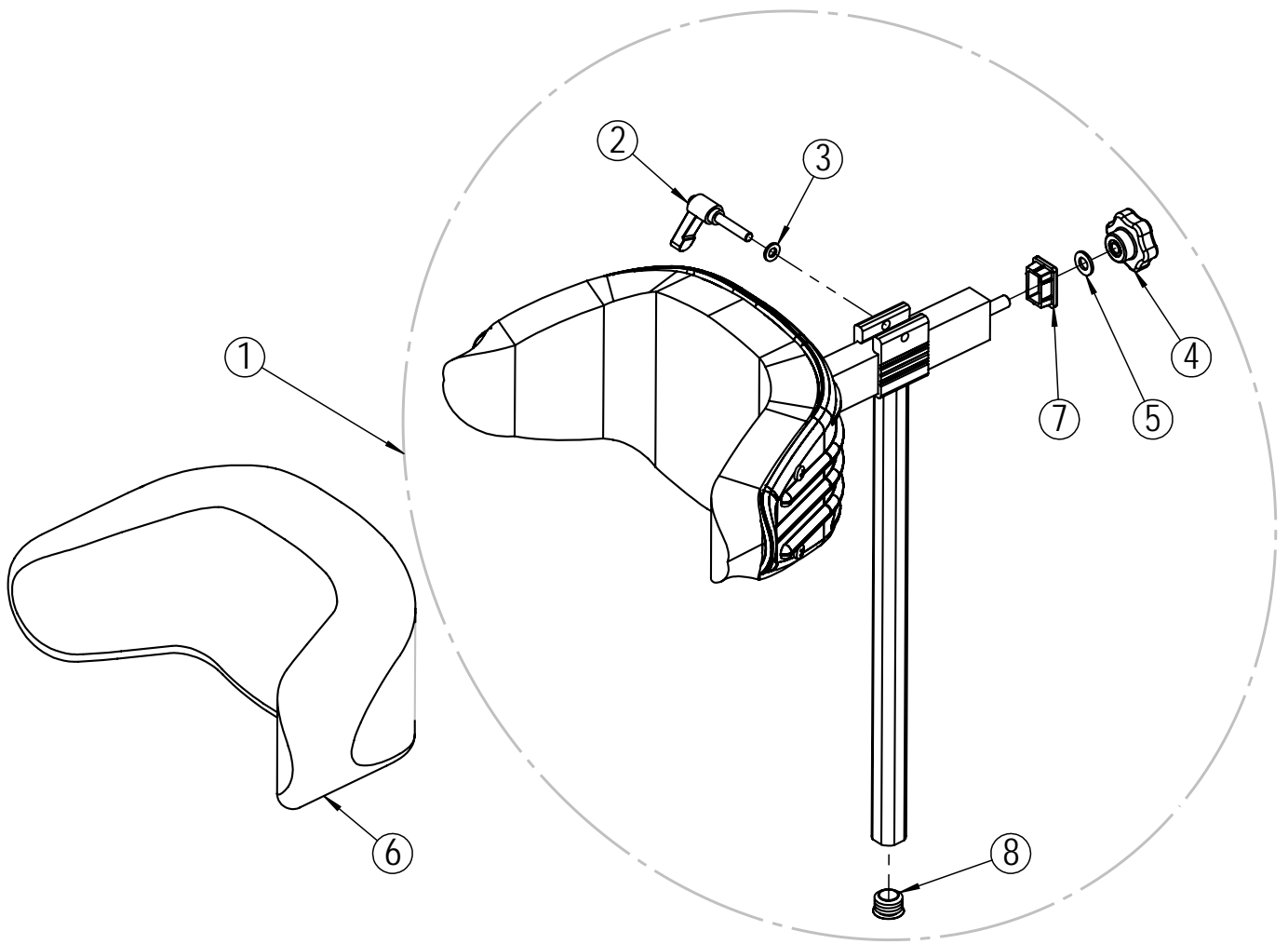

Head support A

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-----------------------------------|----------|----------------|
| 1 | Head support A with Mars | 20575 | 1 |
| 2* | Clamp lever M6x30 | 20282 | 1 |
| 3* | Washer M6 | 20063 | 1 |
| 4* | Star wheel M8 | 26684 | 1 |
| 5* | Washer M8 | 29305 | 1 |
| 6 | Hygiene cover Mars Head support A | 20126 | 1 |
| 7* | Square plug horizontal bar | 20852 | 1 |
| 8* | Plastic plug vertical bar | 20509 | 1 |


* Available upon request.

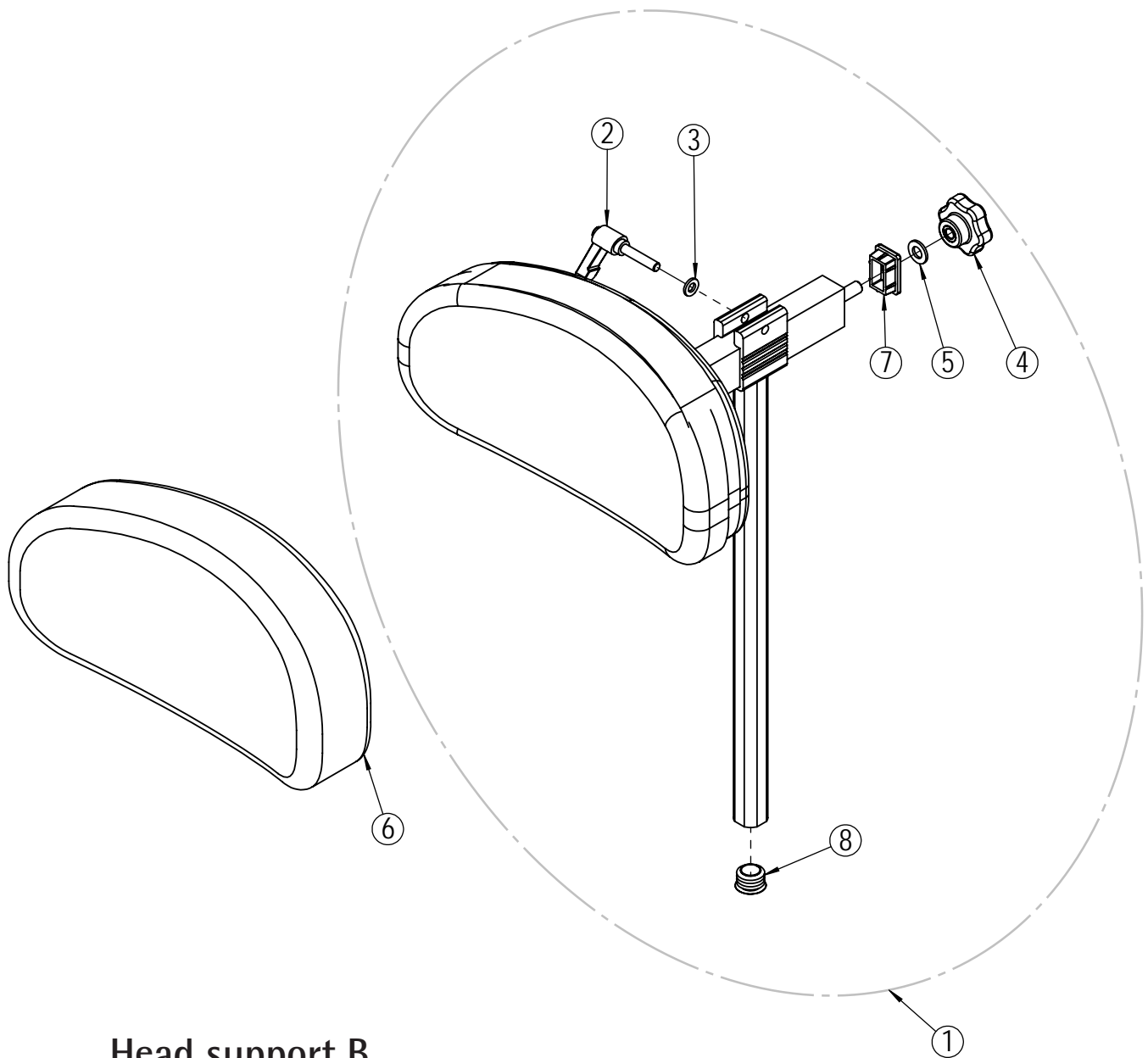

Head support B

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-----------------------------------|----------|----------------|
| 1 | Head support B with Mars | 20574 | 1 |
| 2* | Clamp lever M6x30 | 20282 | 1 |
| 3* | Washer M6 | 20063 | 1 |
| 4* | Star wheel M8 | 26684 | 1 |
| 5* | Washer M8 | 29305 | 1 |
| 6 | Hygiene cover Mars Head support B | 20629 | 1 |
| 7* | Square plug horizontal bar | 20852 | 1 |
| 8* | Plastic plug vertical bar | 20509 | 1 |


* Available upon request.

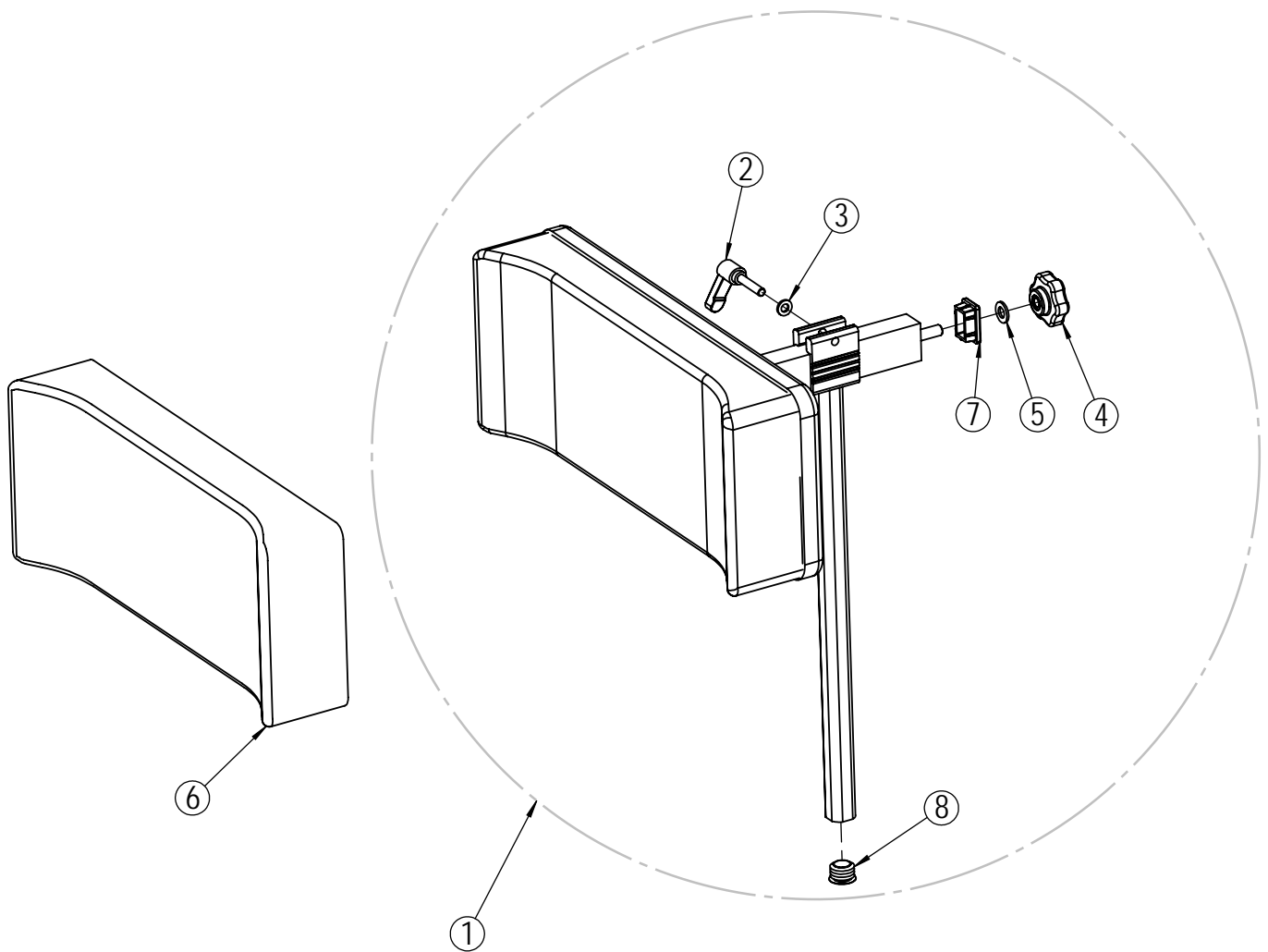

Head support C

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-----------------------------------|----------|----------------|
| 1 | Head support C with Mars | 20573 | 1 |
| 2* | Clamp lever M6x30 | 20282 | 1 |
| 3* | Washer M6 | 20063 | 1 |
| 4* | Star wheel M8 | 26684 | 1 |
| 5* | Washer M8 | 29305 | 1 |
| 6 | Hygiene cover Mars Head support C | 20628 | 1 |
| 7* | Square plug horizontal bar | 20852 | 1 |
| 8* | Plastic plug vertical bar | 20509 | 1 |


* Available upon request.

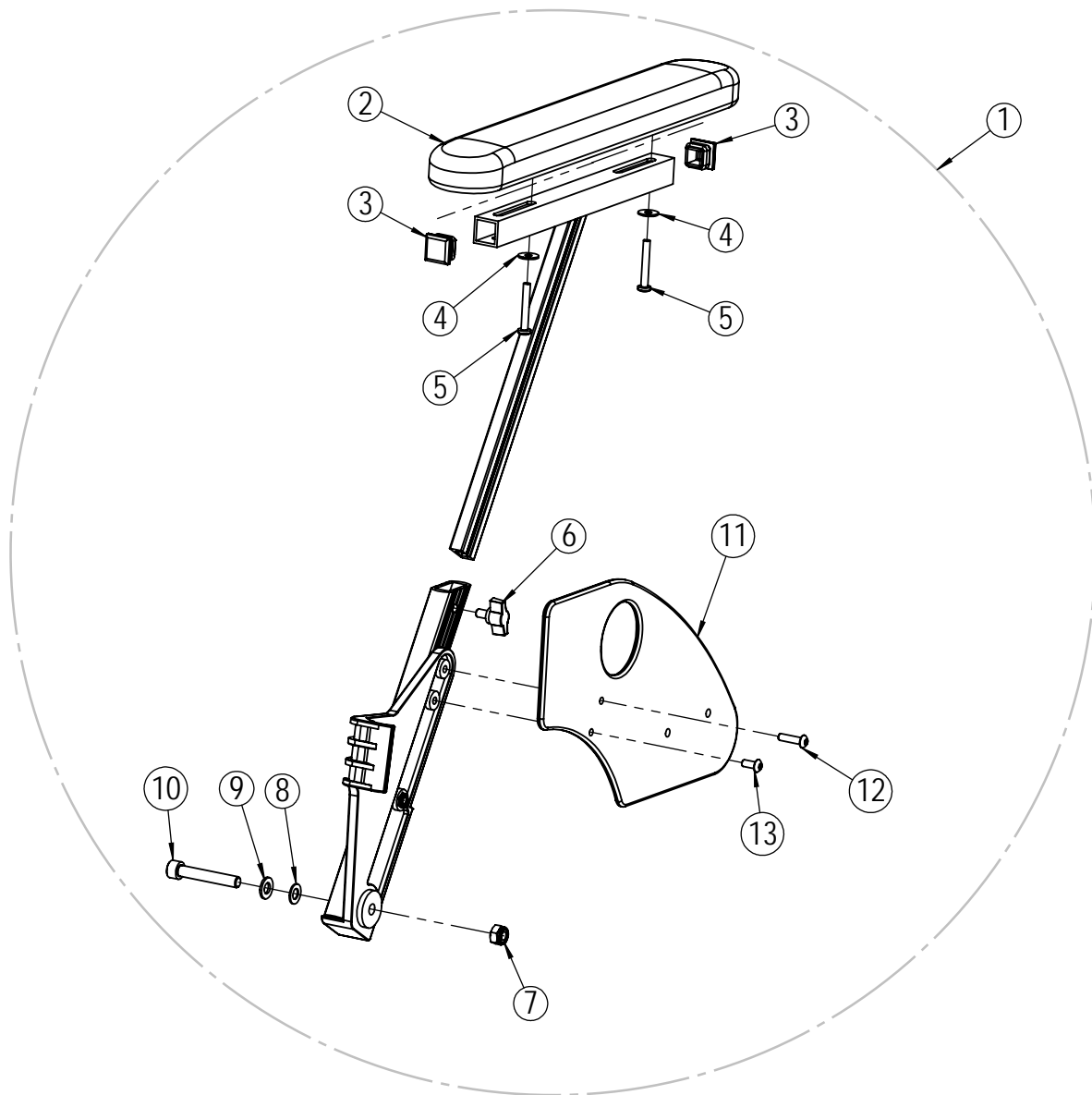

Arm support - left

| Position | Description | Item no. | Pcs. pr. chair |
|----------|---------------------------------------|----------|----------------|
| 1 | Arm support Netti Svipp left complete | 23234 | 1 |
| 2 | Arm support pad 385x58 PU with screws | 20835 | 1 |
| 3* | Square plug | 20633 | 2 |
| 4* | Washer M5 | 23594 | 2 |
| 5* | Screw M5x35 | 21323 | 2 |
| 6* | Wing screw M6x10 | 23256 | 1 |
| 7* | Lock nut M8 | 20412 | 1 |
| 8* | Washer nylon Ø15 | 23362 | 1 |
| 9* | Washer M8 | 20068 | 1 |
| 10* | Screw M8x50 | 25311 | 1 |
| 11* | Skirt guard | 25503 | 1 |
| 12* | Screw M5x20 | 25438 | 1 |
| 13* | Screw M5x12 | 25310 | 1 |


* Available upon request

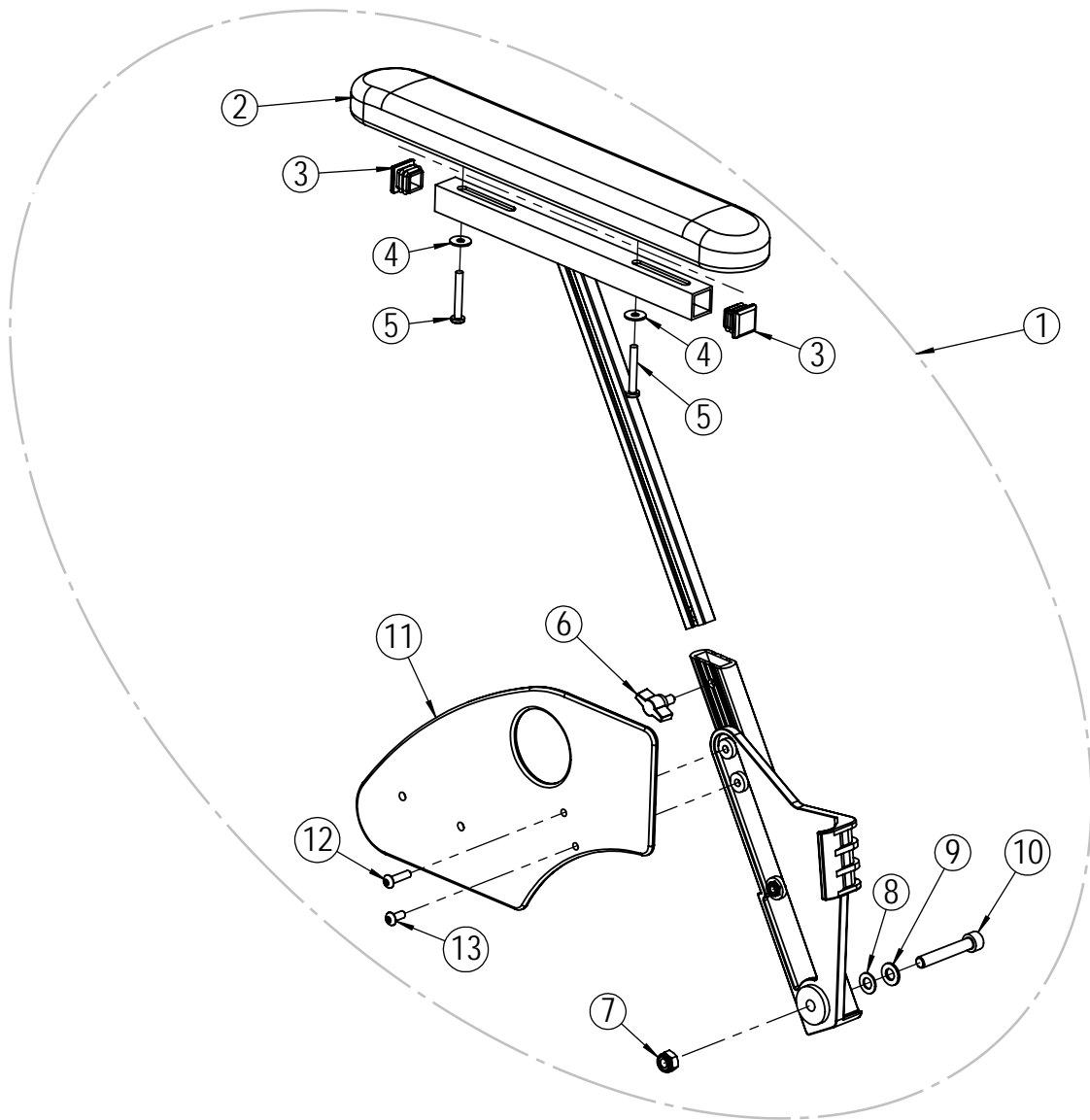

Arm support - right


| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Arm support Netti Svipp right complete | 23226 | 1 |
| 2 | Arm support pad 385x58 PU with screws | 20835 | 1 |
| 3* | Square plug | 20633 | 2 |
| 4* | Washer M5 | 23594 | 2 |
| 5* | Screw M5x35 | 21323 | 2 |
| 6* | Wing screw M6x10 | 23256 | 1 |
| 7* | Lock nut M8 | 20412 | 1 |
| 8* | Washer nylon Ø15 | 23362 | 1 |
| 9* | Washer M8 | 20068 | 1 |
| 10* | Screw M8x50 | 25311 | 1 |
| 11* | Skirt guard | 25503 | 2 |
| 12* | Screw M5x20 | 25438 | 2 |
| 13* | Screw M5x12 | 25310 | 2 |

* Available upon request

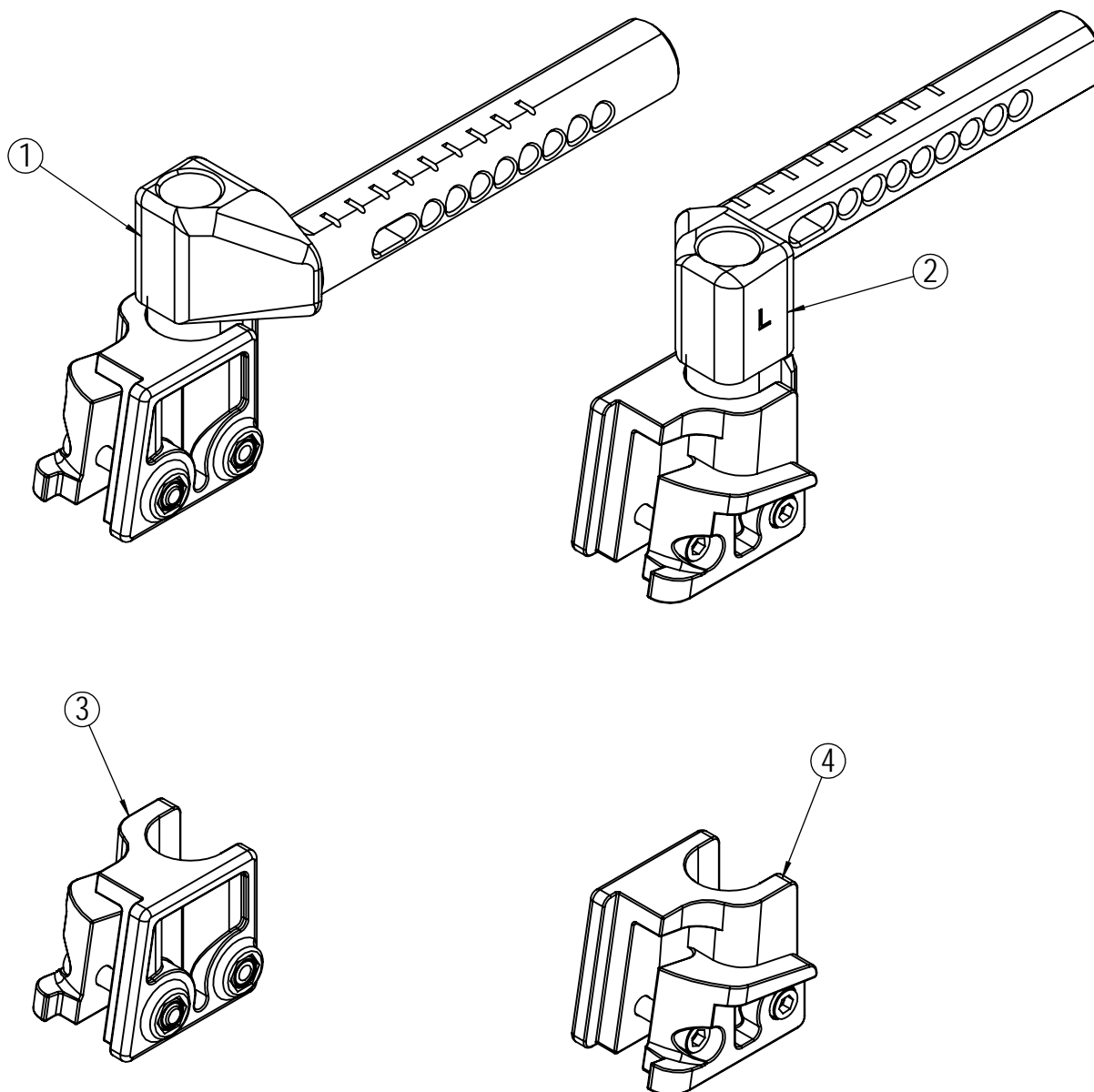


Extension piece - foot support

| Position | Description | Item no. | Pcs. pr. chair |
|----------|---|----------|----------------|
| 1 | Extension piece. Silver. Right. Complete. | 27384 | 1 |
| 2 | Extension piece. Silver. Left. Complete. | 27385 | 1 |
| 3 | Locking bracket with screw. Right. | 29555 | 1 |
| 4 | Locking bracket with screw. Left. | 29556 | 1 |

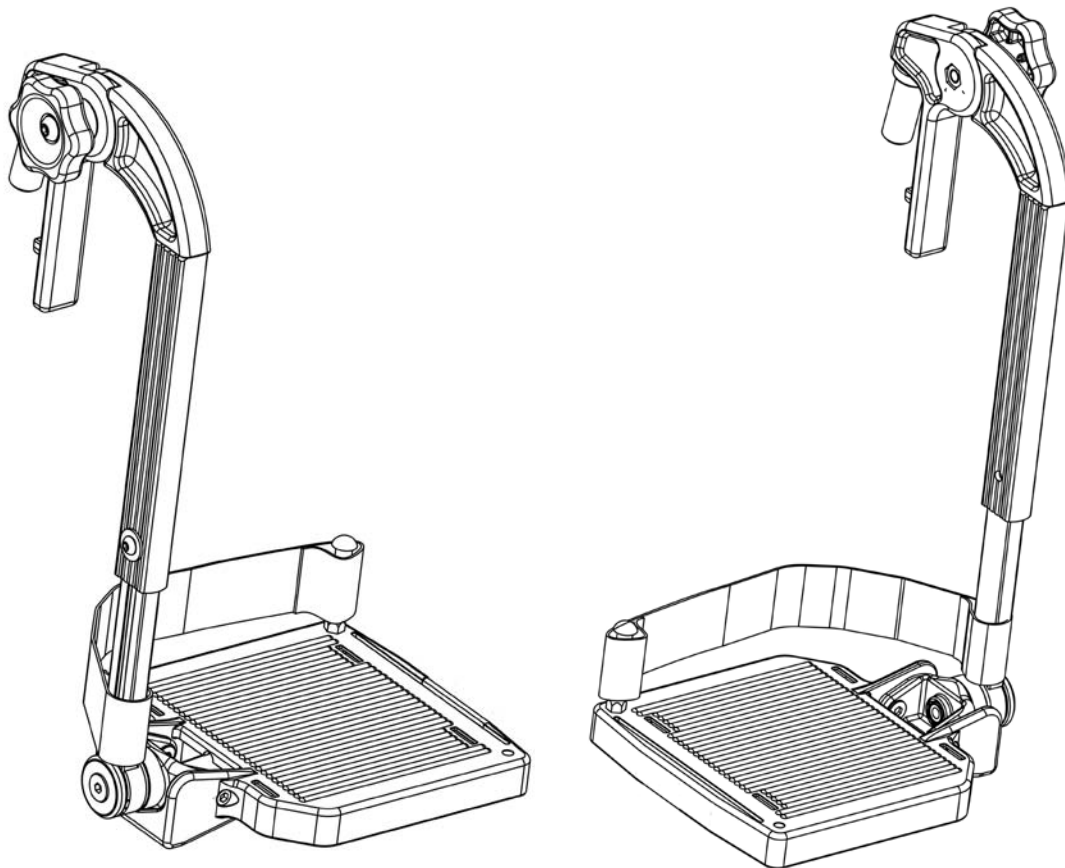

Universal foot supports - complete


| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Universal foot support. Silver. 35cm. Right. | 27220 | 1 |
| 2 | Universal foot support. Silver. 35cm Left. | 27221 | 1 |
| 1 | Universal foot support. Silver. 38cm. Right. | 27222 | 1 |
| 2 | Universal foot support. Silver. 38cm Left. | 27223 | 1 |
| 1 | Universal foot support. Silver. 40cm. Right. | 27224 | 1 |
| 2 | Universal foot support. Silver. 40cm Left. | 27225 | 1 |
| 1 | Universal foot support. Silver. 43cm. Right. | 27226 | 1 |
| 2 | Universal foot support. Silver. 43cm Left. | 27227 | 1 |
| 1 | Universal foot support. Silver. 45cm. Right. | 27228 | 1 |
| 2 | Universal foot support. Silver. 45cm Left. | 27229 | 1 |
| 1 | Universal foot support. Silver. 50cm. Right. | 27230 | 1 |
| 2 | Universal foot support. Silver. 50cm Left. | 27231 | 1 |
| 1 | Universal foot support. Silver. 55cm. Right. | 27232 | 1 |
| 2 | Universal foot support. Silver. 55cm Left. | 27233 | 1 |

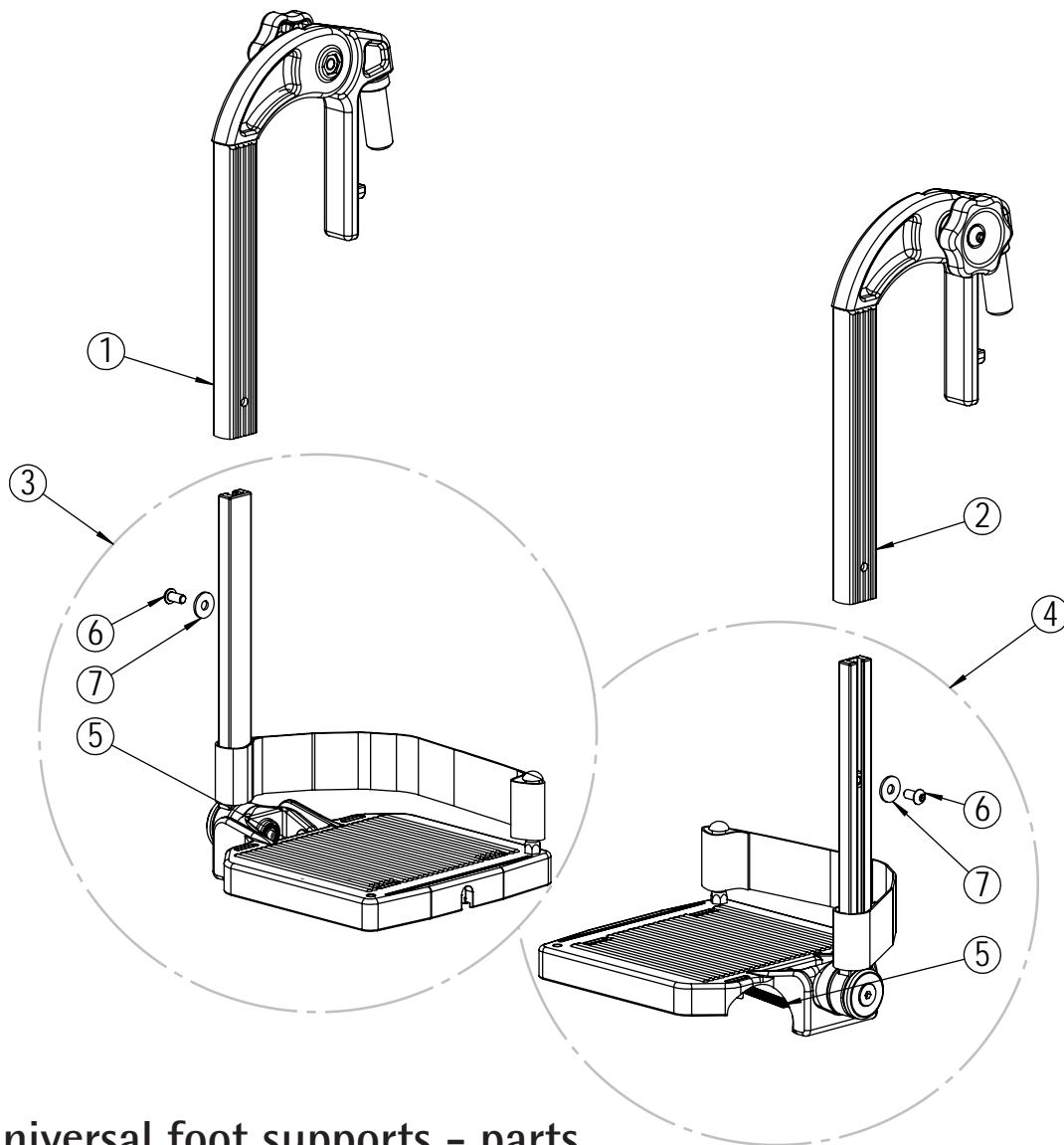

Universal foot supports - parts

| Position | Description | Item no. | Pcs. pr. chair |
|----------|---|----------|----------------|
| 1 | Universal foot support. Swing element. Silver. Right. | 25824 | 1 |
| 2 | Universal foot support. Swing element. Silver. Left. | 26723 | 1 |
| 3 | Footplate with heelstrap. 35cm. Right. | 27875 | 1 |
| 4 | Footplate with heelstrap. . 35cm. Left. | 27899 | 1 |
| 3 | Footplate with heelstrap. . 38cm. Right. | 27876 | 1 |
| 4 | Footplate with heelstrap. . 38cm. Left. | 27900 | 1 |
| 3 | Footplate with heelstrap. . 40cm. Right. | 27877 | 1 |
| 4 | Footplate with heelstrap. . 40cm. Left. | 27901 | 1 |
| 3 | Footplate with heelstrap. . 43cm. Right. | 27878 | 1 |
| 4 | Footplate with heelstrap. . 43cm. Left. | 27902 | 1 |
| 3 | Footplate with heelstrap. . 45cm. Right. | 27879 | 1 |
| 4 | Footplate with heelstrap. . 45cm. Left. | 27903 | 1 |
| 3 | Footplate with heelstrap. . 50cm. Right. | 27880 | 1 |
| 4 | Footplate with heelstrap. . 50cm. Left. | 27904 | 1 |
| 3 | Footplate with heelstrap. . 55cm. Right. | 27881 | 1 |
| 4 | Footplate with heelstrap. . 55cm. Left. | 27905 | 1 |
| 5* | Footplate with heelstrap. fjær 1x6x65 | 27858 | 2 |
| 6* | Screw M6x12 | 24272 | 2 |
| 7* | Washer 6,4x18x1,5 | 20356 | 2 |

* Available upon request

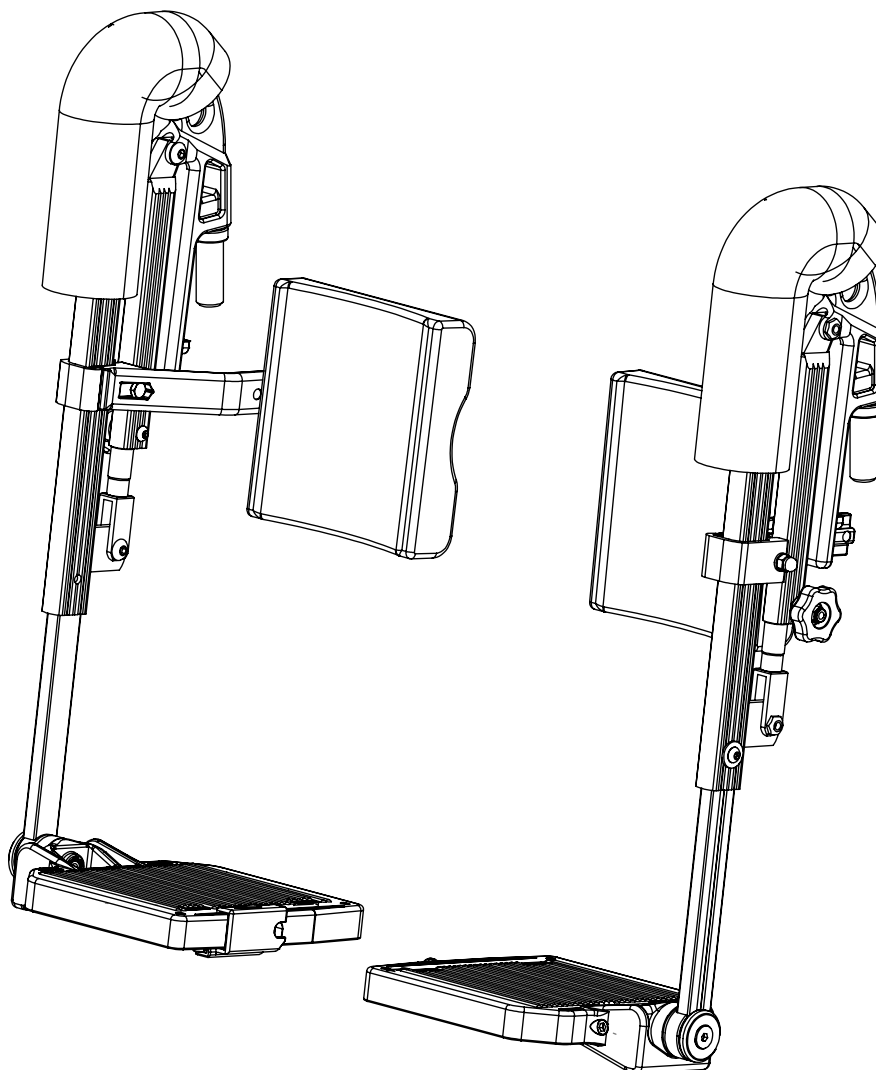


Angle adjustable foot supports - complete


| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Angle adjustable foot support. Black. 35cm. Right. | 29500 | 1 |
| 2 | Angle adjustable foot support. Black. 35cm. Left. | 29508 | 1 |
| 1 | Angle adjustable foot support. Black. 40cm. Right. | 29502 | 1 |
| 2 | Angle adjustable foot support. Black. 40cm. Left. | 29510 | 1 |
| 1 | Angle adjustable foot support. Black. 43cm. Right. | 29503 | 1 |
| 2 | Angle adjustable foot support. Black. 43cm. Left. | 29511 | 1 |
| 1 | Angle adjustable foot support. Black. 45cm. Right. | 29504 | 1 |
| 2 | Angle adjustable foot support. Black. 45cm. Left. | 29512 | 1 |
| 1 | Angle adjustable foot support. Black. 50cm. Right. | 29505 | 1 |
| 2 | Angle adjustable foot support. Black. 50cm. Left. | 29513 | 1 |
| 1 | Angle adjustable foot support. Black. 55cm. Right. | 29506 | 1 |
| 2 | Angle adjustable foot support. Black. 55cm. Left. | 29514 | 1 |

Angle adjustable foot supports - parts

| Position | Description | Item no. | Pcs. pr. chair |
|----------|---|----------|----------------|
| 3 | Angle adjustable foot support. Black. Right. | 29489 | 1 |
| 4 | Angle adjustable foot support. Black. Left. | 29490 | 1 |
| 5 | Knee upholstery. Mars. | 23864 | 2 |
| 6 | Calf pad with bracket. 35cm | 27964 | 1 |
| 6 | Calf pad with bracket. 40cm | 27965 | 1 |
| 6 | Calf pad with bracket. 43cm | 27966 | 1 |
| 6 | Calf pad with bracket. 45cm | 27966 | 1 |
| 6 | Calf pad with bracket. 50cm | 27967 | 1 |
| 6 | Calf pad with bracket. 55cm | 27968 | 1 |
| 7 | Foot plate. 35cm Right. | 28621 | 1 |
| 8 | Foot plate. 35cm Left. | 28649 | 1 |
| 7 | Foot plate. 40cm Right. | 28623 | 1 |
| 8 | Foot plate. 40cm Left. | 28651 | 1 |
| 7 | Foot plate. 43cm Right. | 28624 | 1 |
| 8 | Foot plate. 43cm Left. | 28652 | 1 |
| 7 | Foot plate. 45cm Right. | 28625 | 1 |
| 8 | Foot plate. 45cm Left. | 28653 | 1 |
| 7 | Foot plate. 50cm Right. | 28626 | 1 |
| 8 | Foot plate. 50cm Left. | 28654 | 1 |
| 7 | Foot plate. 55cm Right. | 28627 | 1 |
| 8 | Foot plate. 55cm Left. | 28655 | 1 |
| 9* | Foot plate spring | 27858 | 2 |
| 10* | Screw M6x12 | 24272 | 2 |
| 11* | Washer | 20356 | 2 |
| 12 | Mechanical adjustment piece with screw. Black. Right. | 29558 | 1 |
| 13 | Mechanical adjustment piece with screw. Black. Left. | 29559 | 1 |
| 14* | Screw M6x40 | 20103 | 2 |
| 15* | Adjustment wheel | 27388 | 2 |
| 16 | Foot plate hinge complete | 28523 | 2 |
| 17 | Calf support bracket with screw, washer and lock nut | 25353 | 2 |
| 18* | Nut M8 | 29761 | 2 |
| 19* | Foot support locking cone | 20539 | 2 |
| 20* | Nut M6 | 20058 | 2 |

* Available upon request.

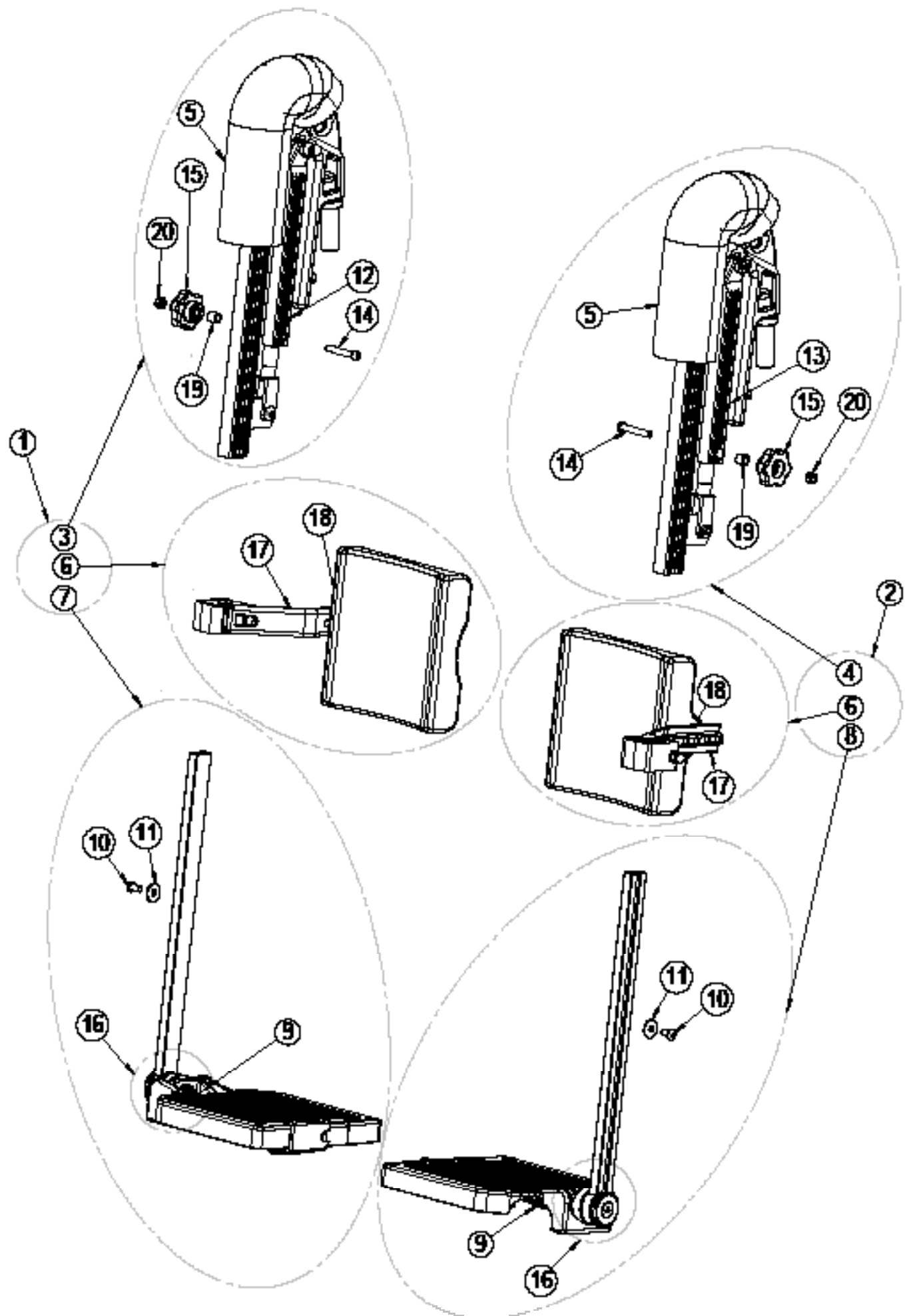


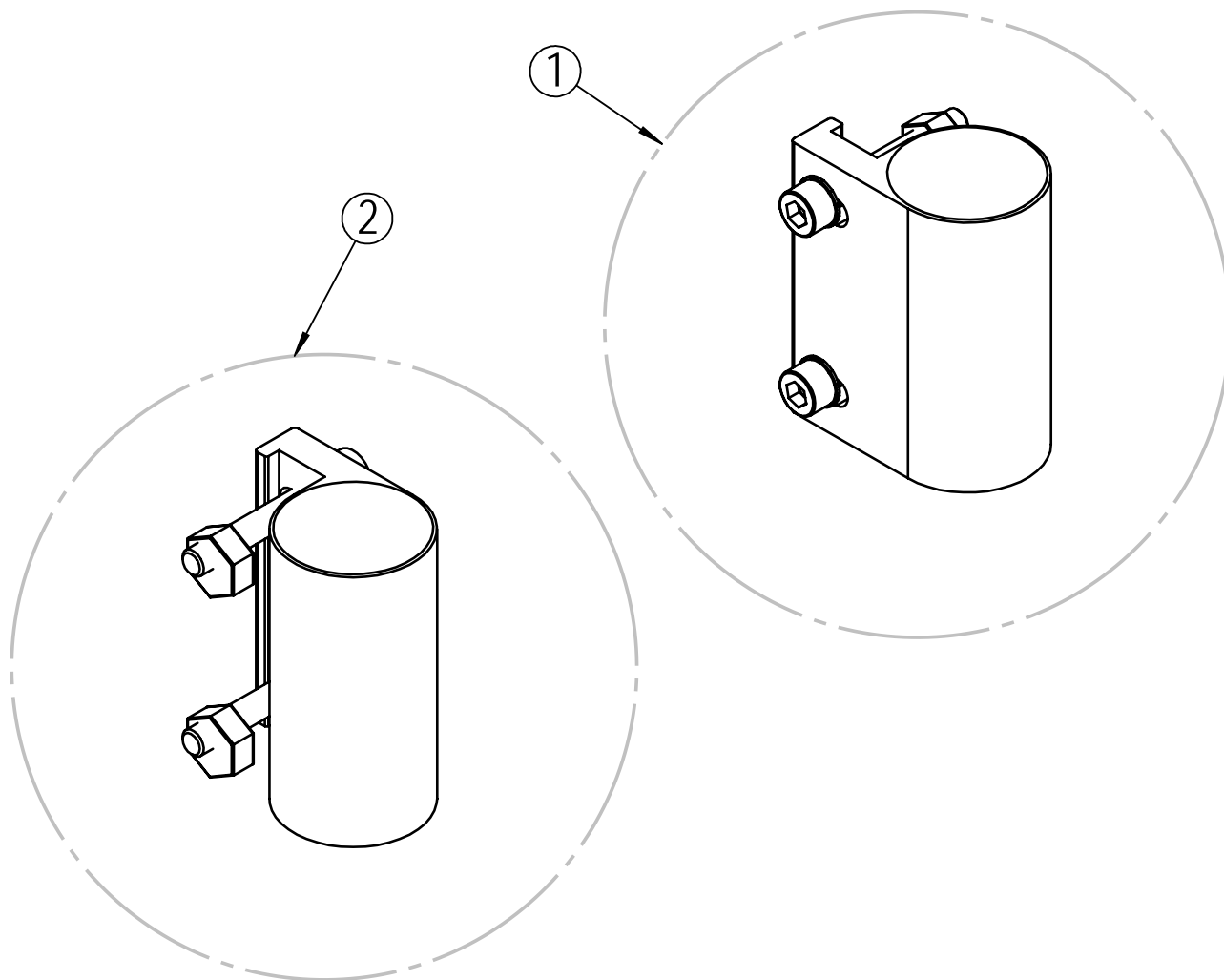

Bearing house

| Position | Description | Item no. | Pcs. pr. chair |
|----------|--------------------------|----------|----------------|
| 1 | Bearing house A complete | 26445 | 1 |
| 2 | Bearing house B complete | 26446 | 1 |


* Available upon request

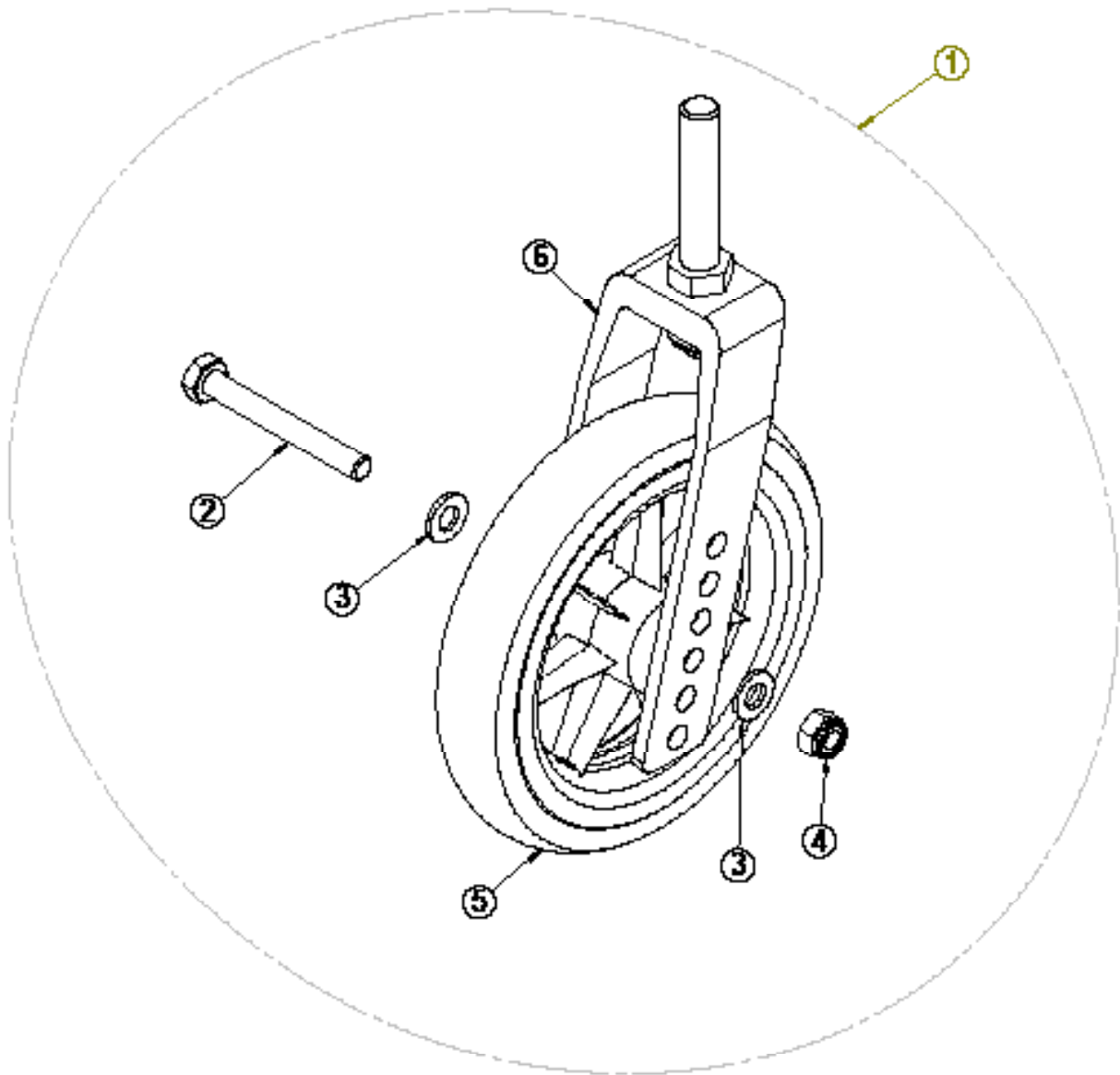

Front castors


| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Fork standard with 6" front castors complete | 20538 | 2 |
| 1 | Fork standard with 7" front castors complete | 20535 | 2 |
| 2* | Screw M8x65 | 20419 | 2 |
| 3* | Washer M8 | 20068 | 4 |
| 4* | Lock nut M8 | 20412 | 2 |
| 5 | Front castor 6" | 20817 | 2 |
| 5 | Front castor 7" | 20976 | 2 |
| 6 | Front castor standard with axle | 20429 | 2 |

* Available upon request.

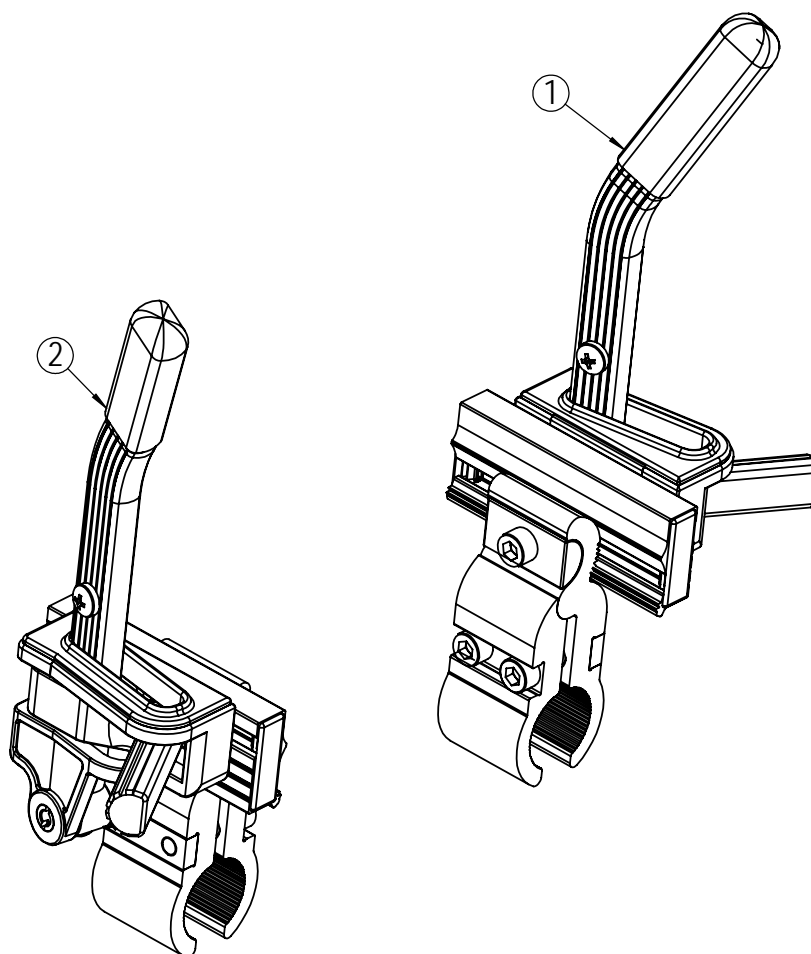


Brake

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-------------------------------------|----------|----------------|
| 1 | Brake Universal Netti I/Svipp Left | 26824 | 1 |
| 2 | Brake Universal Netti I/Svipp Right | 26817 | 1 |

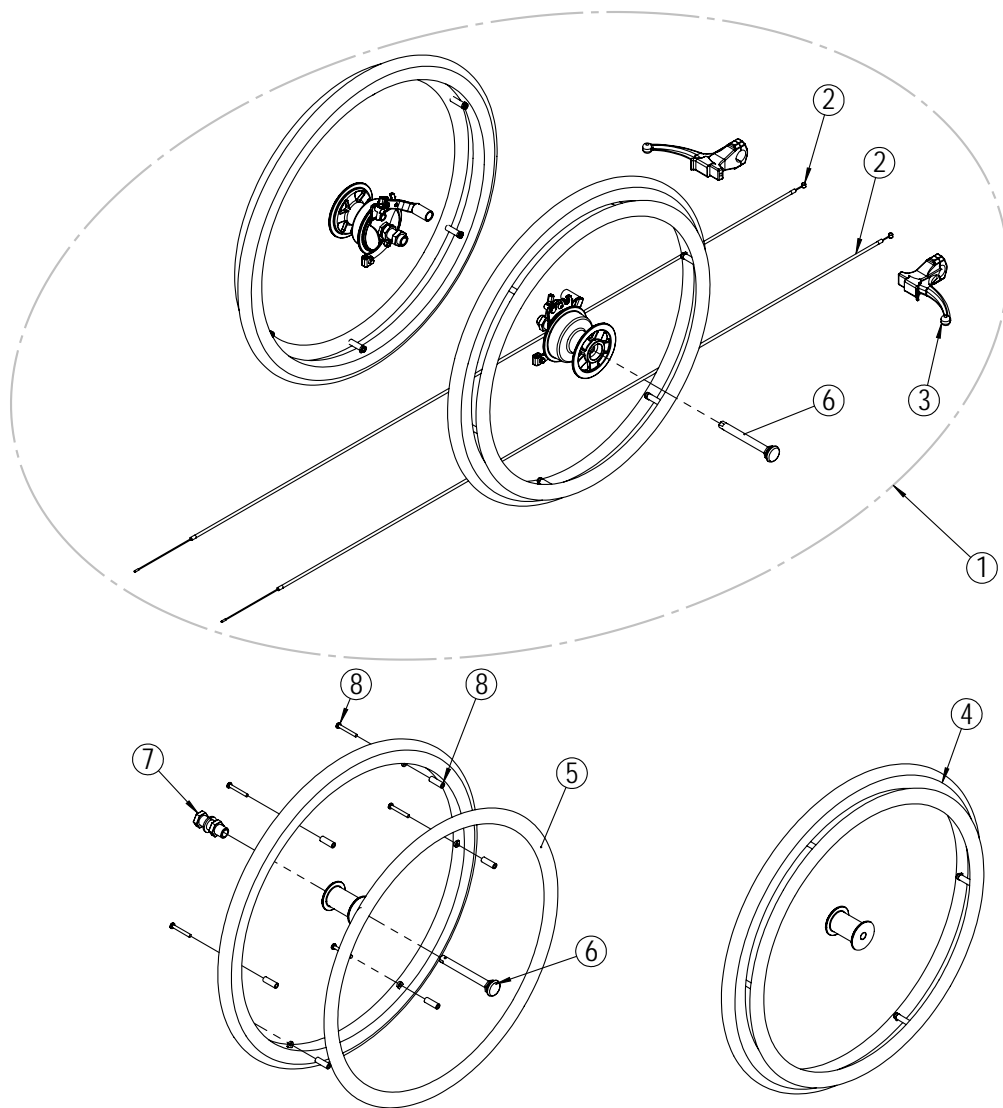

Main wheels

| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Main wheel 22" x 1" with drum brake - kit | 24365 | 1 |
| 1 | Main wheel 24" x 1" with drum brake - kit | 24310 | 1 |
| 2 | Wire 1250 mm complete | 41310 | 2 |
| 3 | Brake handle | 41313 | 2 |
| 4 | Main wheel 22" x 1" with quick release | 25371 | 2 |
| 4 | Main wheel 24" x 1" with quick release | 25372 | 2 |
| 5 | Push rim 22" plast | 20451 | 2 |
| 5 | Push rim 24" plast | 20450 | 2 |
| 6 | Quick release Ø12 x 98mm | 20449 | 2 |
| 7 | Wheel bushing complete | 20329 | 2 |
| 8 | Screw (6 pcs.) & distance sleeves (6 pcs.) - kit | 21111 | 2 |

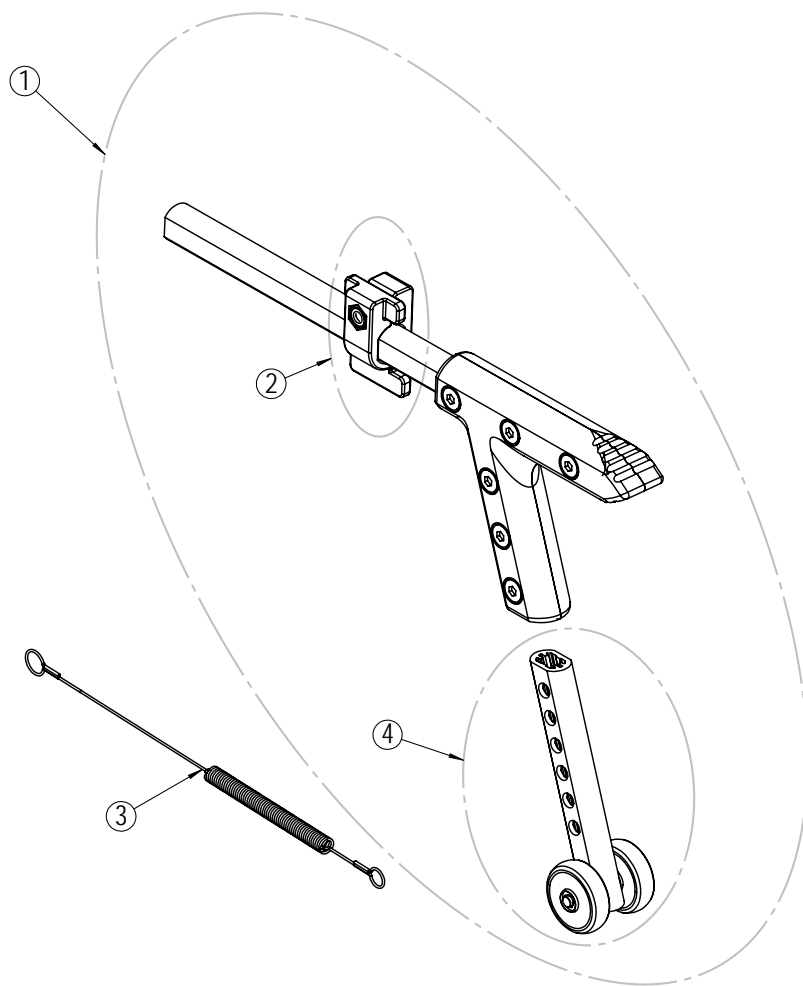

Anti tip

| Position | Description | Item no. | Pcs. pr. chair |
|----------|--|----------|----------------|
| 1 | Anti tip long/short | 25419 | 2 |
| 2 | Anti tip lock bracket complete | 26980 | 2 |
| 3* | Anti tip spring 445mm | 24242 | 2 |
| 4* | Anti tip vertical bar with wheels standard | 26118 | 2 |

* Available upon request